AEROprotective - Filterierende Halbmaske FFP2 NR Respiratory Face Mask FFP2 NR

ID 4372426 Rev. 1.0 | 19.10.2020

① DE: Maskenkörper, EN: Face mask body, FR: Corps du masque, IT: Corpo dalla maschera, ES: Cuerpo de la mascarilla, PT: Corpo da máscara. ② DE: Ohrschlaufe, EN: earloop, FR: Élastiques pour les oreilles, IT: Elastici per le orecchie, ES: Elásticos para las orejas, PT: Elásticos para as orelhas. ③ DE: Nasenbügel, EN: Nose clip, FR: Clip du pont de nez, IT: Clip nasale, ES: Pinza nasal, PT: Clipe de nariz. ④ DE: Ohrenschnur, EN: Earloop buckle, FR: Crochet pour élastiques, IT: Gancio per elastici, ES: Gancho para elásticos, PT: Gancho para elásticos. ⑤ DE: Schaumabdichtung, EN: Sponge pad, FR: Mousse d'étanchéité, IT: Sigillatura in schiuma, ES: Sellado de espuma, PT: Vedação de espuma.

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Product Name Respiratory Face Mask FFP2 NR

Model M-9520

Intended Use As a respirator, it is intended to help reduce wearer exposure to certain airborne particles (solid and liquid aerosols) which are hazardous to health.

Reference standard EN 149:2001+A1:2009

Classification FFP2 NR

Reference regulation Regulation (EU) 2016/425 of the European Parliament and of the Council

Components Face mask body, nose clip, ear loop, ear loop buckle, sponge pad, ear loop

Material Polypropylene non-woven fabrics (face mask body)

Specs acc. EN 149:2001+A1:2009

- Maximum penetration of test aerosol: $\leq 6\%$
- Total inward leakage (TIL): $\leq 8\%$
- Inhalation resistance: ≤ 70 Pa at 30 l/min, ≤ 240 Pa at 95 l/min
- Exhalation resistance: ≤ 300 Pa at 160 l/min

Checks prior to use Before use, check the shelf life of the mask and the technical condition whether the mask has no visible mechanical damage, is not contaminated or incomplete. A damaged mask and the one whose storage conditions have been exceeded cannot be used.

Usage Method (see also illustrations above) 1. Pull an extend the mask inner side to the face with two hands, ensure the nose clip is upward. 2. Hold the mask on the face and pull the earloops to the ears. (Attached to one of the earloops there is a buckle which can be used to connect the two earloops behind the head to relieve pressure from the ears. This can be especially useful when the mask is worn during extended periods of time or on a daily basis. If the buckle is not used, it can be easily removed from the earloop.) 3. Press the nose clip to fit the shape of the nose. 4. Check and press the mask to fit the face to minimize leakage. If you cannot achieve a proper seal, DO NOT enter the contaminated area. See your supervisor.

Wearing duration Less than 8 hours is recommended.

Limits of use and protection The product is non-sterile, please don't use in environment sterile required.

Warning information 1. This respirator is limited to single shift use only. 2. This respirator helps protect against certain particulate contaminants but does not eliminate exposure to or the risk of contracting any disease or infection. Before any use please read carefully the instructions for use and keep it. Never use the product outside its intended area of use. It is brought to the attention of users that a product, even tested and certified according to one or more standards, is never a total guarantee of protection against the risks involved. Always read the limits of use and protection and the warning information. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death. For proper use, see supervisor, or user instructions, or contact occupational safety for instruction. 3. This respirator does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. 4. Do not use when concentrations of contaminants are immediately dangerous to health or life. 5. Do not use in explosive atmospheres. 6. Leave work area immediately and return to fresh air if a) breathing becomes difficult, or b) dizziness or other distress occurs. 7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator. 8. Never alter or modify this respirator in any way.

Shelf life Three Years

Storage condition Store in a dry place and at room temperature, away from light.

Manufacturer Company: Trust Loyalty & Glory Epoch Network Technology Co., Ltd. Address: No. 39 Jianguo

Road, Suzhou Industrial Park, Suzhou City, Jiangsu Province, China Website: <http://www.mainage.com/>

EU Authorized Representative Name: Lotus NL B.V. Address: Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands E-mail: peter@lotusnl.com

The notified body involved in the conformity assessment and the notified supervisory body: ÖTI - Institut für Ökologie, Technik und Innovation GmbH A-1050 Vienna, Spengergasse 20 Notified Body No. NB 0534

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